

Sonic Irrigator® PCF_{system}®

Medisafe SI PCF® – Instructions for Use with da Vinci® EndoWrist® Instruments*

*For complete listing of da Vinci® EndoWrist® instruments validated with the Medisafe SI PCF® system, visit www.intuitivesurgical.com

- 1 Scrub the outside of the entire instrument using running water with a clean, soft, nylon bristled brush, paying special attention to the instrument tip. Move the instrument wrist through its full range of motion while scrubbing. *Further information for this may be found in the EndoWrist® Instruments IFU (Instructions for Use).*
 - 2 Thoroughly rinse the entire instrument especially the instrument tip.
 - 3 Connect the instrument to the Medisafe SI PCF® “Intuitive® Basket” ONLY. This is clearly marked as ‘For Intuitive Instruments only’.
 - 4 Attach the ‘slip luer’ end of the Medisafe connection tubing to the main flush port of each instrument by firmly pushing and twisting to ensure that the luer is firmly seated. The main flush port is designated by a dark ring and the number 1.
- ⚠ WARNING:** In order for the instrument to be processed properly, the Medisafe SI PCF® system **MUST** be connected to the main flush port on the EndoWrist® instrument. Refer to EndoWrist® Instrument IFU for additional details on the location of the flush ports.
- 5 Connect no more than 10 instruments at any time to the Medisafe SI PCF® “Intuitive® Basket”
 - 6 Place the basket into the Medisafe SI PCF® system – once inserted move the basket to the right until arrows on the basket and the back of the tank are in line.
 - 7 Once in line, insert the black basket lock device into one of the four basket guide posts.
 - 8 Once fully secured, close the lid placing slight force on the front left to ensure lid is locked.
 - 9 Programme selection - **Select programme 3** for ‘da Vinci’ wash cycle only or **programme 6** for ‘da Vinci’ wash cycle with thermal disinfection.
 - 10 Press the **START** button on the control panel to start the ‘da Vinci cycle’.
 - 11 Once the cycle has finished ‘**UNLOAD INSTRUMENTS**’ will be displayed.
 - 12 Remove the instruments from the machine.
- ⚠ CAUTION –** If any instruments have detached from the connection tubing during processing, they should be reprocessed ensuring that the luer connection is firmly seated. If programme 6 has been selected the EndoWrist® instruments and basket may be hot.
- 13 Dry the instrument completely. Ensure that all water is emptied from the instrument shaft and main flush port by vertically positioning the instrument with tip up while drying. Dry the outside of the instrument with a lint-free cloth. Air may be blown through all flush ports to facilitate drying.
 - 14 Lubricate the tip and wrist mechanism with a neutral pH, steam-permeable instrument lubricant per the manufacturer’s instructions. Proceed to sterilization.



Medisafe International
Twford Road, Bishop’s Stortford,
Hertfordshire, CM23 3LJ, UK
Telephone
+44 (0)1279 461641

Medisafe America, LLC
4710 Eisenhower Blvd,
Suite B-4, Tampa FL 33634
Call Toll Free
877-444-6334

